



Thank you for purchasing the NRG Grasshopper Chair Package. This professional massage chair will provide you with years of performance on the road or in your treatment room.

ASSEMBLY INSTRUCTIONS

1. Remove the chair from the box and open the carry case.
2. Unfold the front legs to open the chair.
3. Lift the seat and place the bar into the appropriate height slot.
4. Release headrest locking arm and unfold headrest.
5. Adjust the headrest height and angle for client comfort. Lock by pushing down on the locking arm.
- Caution – if the headrest side arms are not fully tightened and locked prior to use, the gears may strip when the headrest is used.
6. Lift the arm rest and firmly place armrest locking arm into one of the 3 adjustment positions.
7. Loosen chest plate and adjust to the right height for your client. Tighten the knob to lock in place.

CLEANING

Clean upholstery with mild soap and water. If using a multi-purpose cleaner test in an inconspicuous spot to ensure there is no damage to the vinyl or degradation of color.

WARRANTY

1 Year limited warranty from date of purchase.

