




Earthlite






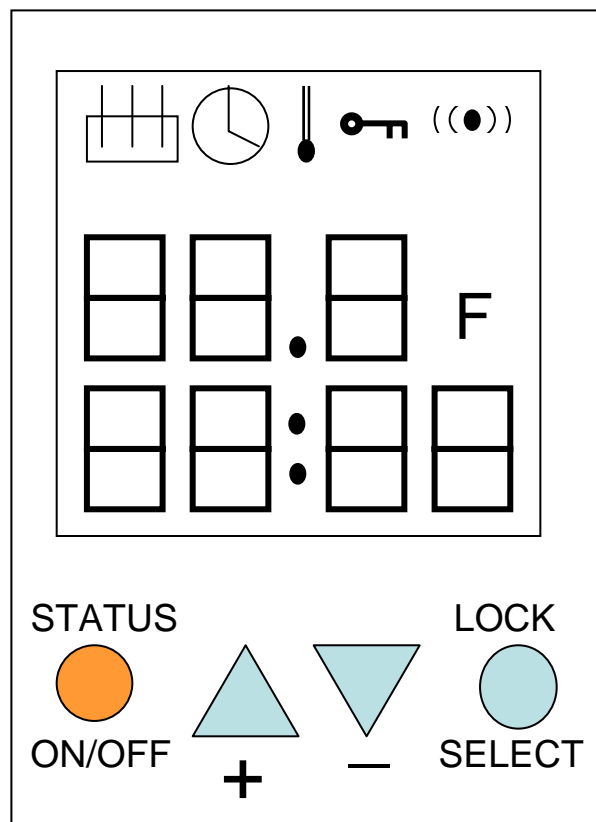

Samadhi-Pro

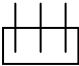
Table Warmer

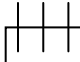

Press and hold “orange circle” button to turn it on and off.

When turned on, you will see  on top of the screen. This indicates the unit is on and working properly. If this is the only thing shown on the screen, it means that the screen is showing status - current temperature (middle) and counting how long it's on (bottom). If  is shown, it means the unit is not working properly.

Press “orange circle” button to switch between “status” and “change setting” modes. If you see  or , along with , it means you are looking at a setting mode (for temperature and period of time). During  is shown, you can set how long the unit stays on by using “+” or “-” blue buttons. During  is shown, you can set temperature. You can switch between  and  by pushing “blue circle” button.

Once you set the ideal time and temperature, you can go back to “status” screen by pressing “orange circle” button (and check your current temperature and time is), or press and hold “blue circle” button to “lock” your setting. Once locked, you will see  on top of the screen. Press and hold “blue circle” button again to unlock.