

# AUTEX RTU RP DEVELOPER REPLENISHER

*For use in automatic processing of compatible processed X-Ray films*

**Catalog Number: 23414 (4 x 1 G)**

## DESCRIPTION

Autex RTU (ready-to-use) RP Developer delivers superior performance and convenience as a one part liquid concentrate. Suitable for all types of films and processors, this product requires no mixing when used as a replenisher.

One box of **23414** contains four (4) bottles of RTU (ready-to-use) concentrate to make a total of 4 gallons or 15.14 liters.

## SAFE HANDLING INSTRUCTIONS

Please refer to *Safety Data Sheets (SDS)* for specific chemical details. Copies in Spanish can be obtained through your FUJIFILM HUNT local distributor.

## SPECIFIC GRAVITY & pH

	pH @ 25° C	SpG @ 25° C
Developer Replenisher	10.50 ± 0.05	1.080 ± 0.005

## SPECS & RECOMMENDATIONS

Temperature	32°C - 35°C
Processing Time	90 - 360 Seconds

## MIXING INSTRUCTIONS:

**Solution is Ready-to-Use (RTU).**

**Mixing is not required for Replenisher**

## MIXING INSTRUCTIONS for WORKING TANK

W	STARTER (29020)	DEVELOPER REPLENISHER
1 Liter	23 ml	1000 ml
1 Gallon	3.0 oz	1 Gallon

## SHELF LIFE

	Concentrate	Mixed Repl.	Working Tank
Time	U = 12 Months O = 2 Weeks	4 Weeks	2 Weeks
Color	Pale Yellow	Pale Yellow	Pale Yellow
Odor	Slight Amino	Slight Amino	Slight Amino

U = Unopened, O = Opened